

FAWSsit™ Portable Shower

ROUTINE CARE AND MAINTENANCE INSTRUCTIONS

Prevent bacteria and mildew formation within the shower unit

AFTER EACH USE:

- Remove solid materials and soap scum from the pan.
- Use a small amount of anti-bacterial soap and warm water to wipe out the shower pan and then rinse with clean water.
- Drain water from the hose before hanging it back on the frame.
- Dry the curtain and pan before folding the unit for storage.

BI-MONTHLY CLEANING ROUTINE

- With the pump off, pour approximately 2-3 gallons of warm water into the shower pan and add approximately ½ cup of household chlorine bleach, or your choice of bleach alternative, to the water.
- Let the solution sit in the pan for approximately 5-10 minutes.
- Use a cloth and the solution from the pan to wipe down the curtains and frame.
- Turn on the pump to circulate the solution through the hose and pump for disinfection and to drain the shower pan.
- To remove the bleach residue, fill the shower pan with clean warm water and use a damp cloth to wipe down the curtains and frame.
- Turn on the pump to circulate the clean water through the hose and pump and drain the shower pan.
- Turn off the pump and dry any remaining moisture on the curtains and pan, drain the hose, then fold the shower for storage.

QUESTIONS? REPAIRS? REPLACEMENT PARTS?

CALL: 1-877-329-7748